
- | | |
|-----------------------|----------------------|
| NL Externe katheters | I Cateteri esterni |
| D Externe Katheter | N Ufvendige katetre |
| F Sondes externes | OK Ufvendige katetre |
| EN External catheters | S Externa katetrar |
| ES Catéteres externos | |

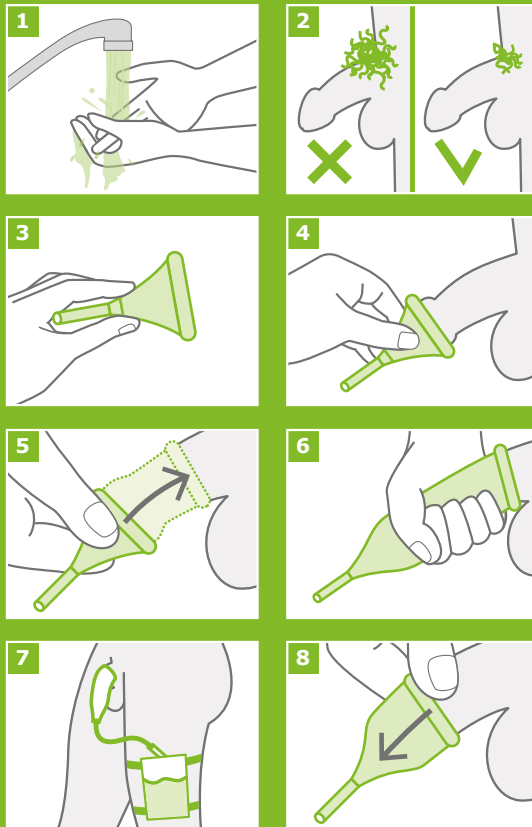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EN Hekura external catheters – Instructions for Use

Important

- For single use.
- Do not store in direct sunlight, at high temperatures or in humid conditions.
- Do not use if the packaging is open or damaged.
- Store the packaging between 10°C and 30°C.
- Use for a maximum of 24 hours.
- Choose the correct size and type together with your healthcare provider. An incorrect size may cause leakage or constriction.

Instructions for Use

- 1 Wash and dry your hands and your penis.
- 2 If necessary, trim your pubic hair.
- 3 Remove the external catheter from its packaging and release it from the holder by carefully pressing the holder with two fingers.
- 4 Grasp the external catheter 1 to 2 cm below the place where the catheter narrows and place it over your penis from there. This will allow the urine to flow freely.
- 5 Then roll the catheter out over the length of your penis.
- 6 Gently press the catheter so it fits comfortably on your skin.
- 7 Close the valve on the urine drainage bag. Attach the connector of the urine drainage bag to the external catheter. Make sure the connection is free of kinks. Use fixation material or a hanging bag holder to secure the urine drainage bag.
- 8 Disconnect the urine drainage bag, then remove the external catheter by rolling it gently off your penis. You can use a washcloth with warm water to make this more easy.

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